




STRONG AND BEAUTIFUL

## CHAIRSIDE PREPARATION GUIDE

JENSEN XT ZIRCONIA


### ANTERIOR CROWN

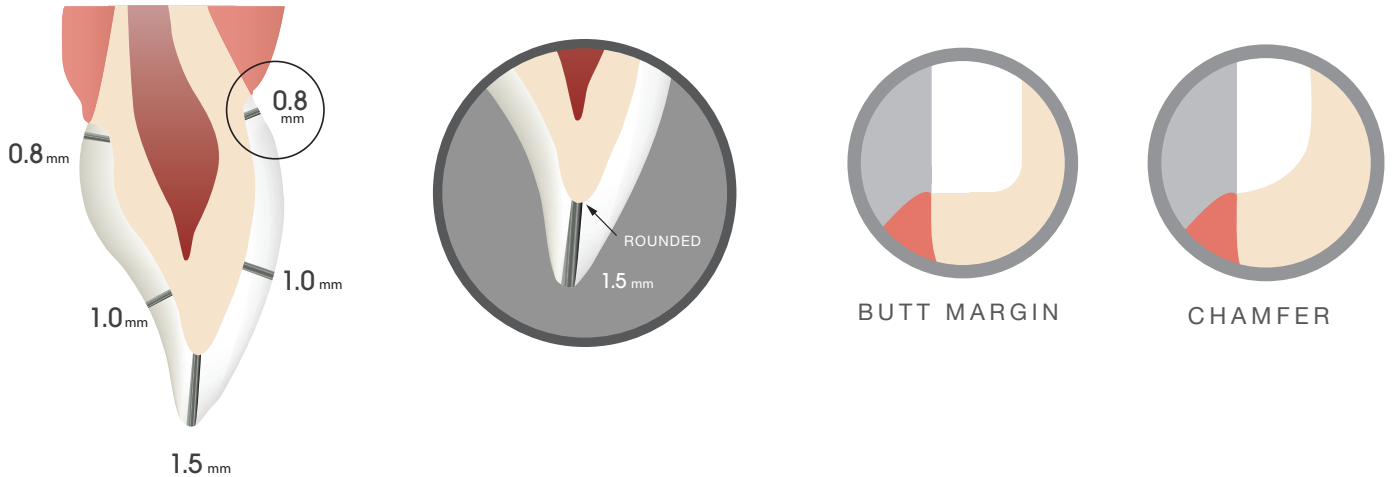



### POSTERIOR CROWN



### INLAY

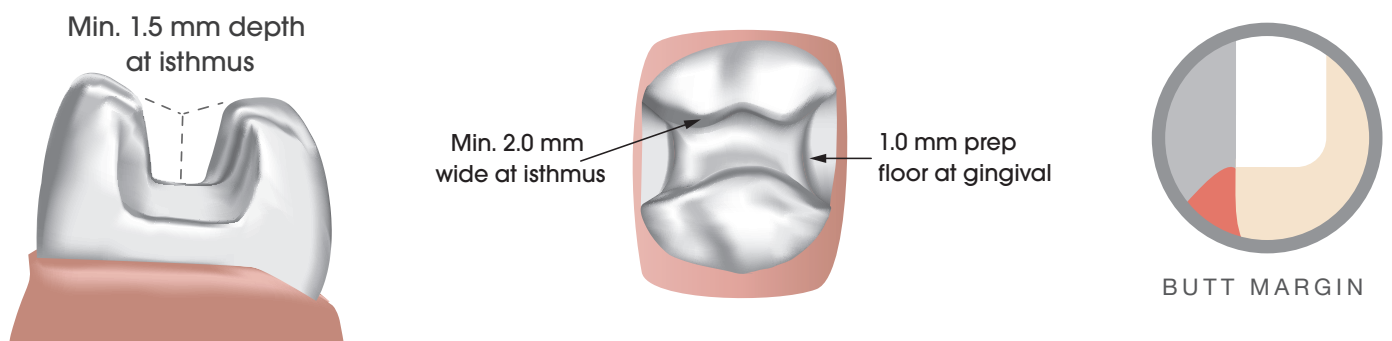

### ONLAY/PARTIAL CROWN

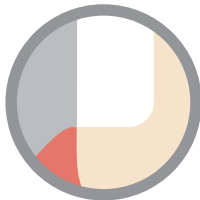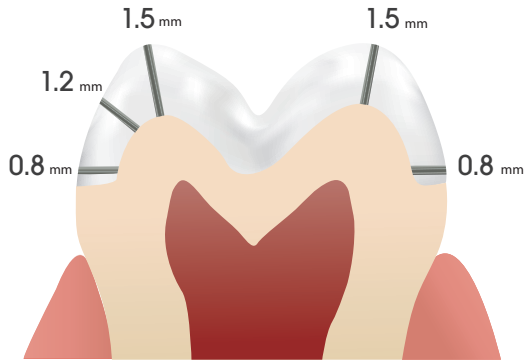

BUTT MARGIN

### CEMENTATION

Conventionally cement following the manufacturer's instructions for use with zirconia restorations.

### CHAIRSIDE ADJUSTMENTS

If chairside adjustments are needed:

- Fine diamond tools are recommended using a water-cooled, high speed hand piece.
- Use light pressure to avoid overheating and micro-cracking of the sintered body.
- Polish with diamond impregnated rubber polishers; maintain minimum wall thickness and connector cross sections.

### INDICATIONS FOR USE

		XT Zirconia
Anterior	Crown	✓
	Bridge*	✓

		XT Zirconia
Posterior	Crown	✓
	Bridge*	✓
	Inlay	✓
	Onlay	✓

\*Bridges up to 16 units

### CONNECTOR CROSS SECTIONS

		XT Zirconia
Anterior	Abutment to Pontic	≥7 mm <sup>2</sup>
	Pontic to Pontic	≥12 mm <sup>2</sup>
	Cantilever Pontics	≥8 mm <sup>2</sup>

		XT Zirconia
Posterior	Pontic to Pontic	≥12 mm <sup>2</sup>
	Abutment to Pontic	≥9 mm <sup>2</sup>
	Abutment to Abutment	≥9 mm <sup>2</sup>
	Cantilever Pontics	≥12 mm <sup>2</sup>
	Inlay Retainer to Pontic	≥9 mm <sup>2</sup>