



ANTERIOR CROWN

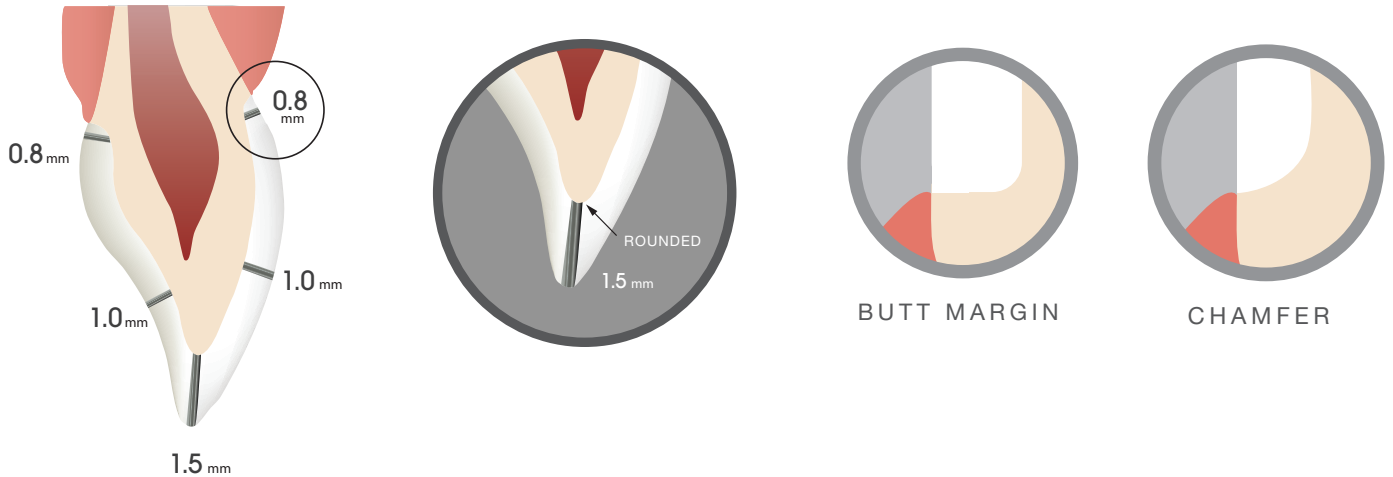


POSTERIOR CROWN



INLAY

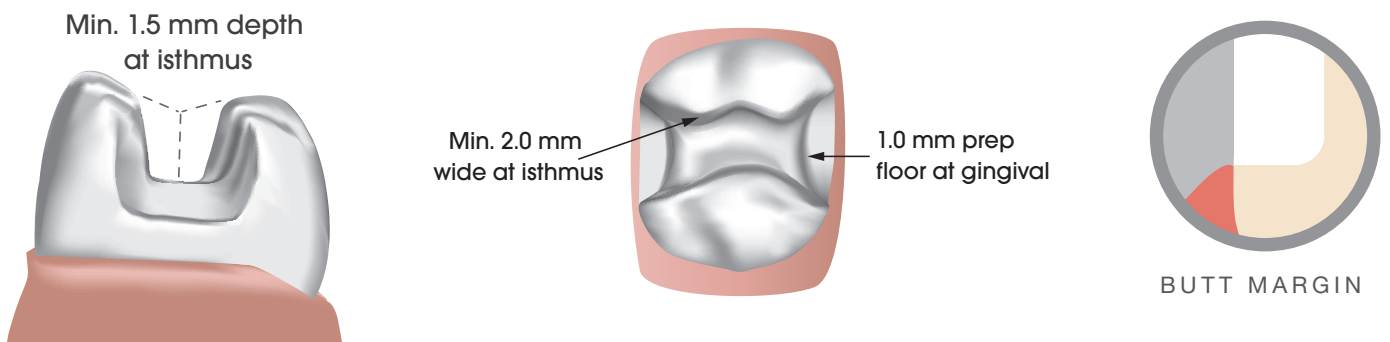

ONLAY/PARTIAL CROWN

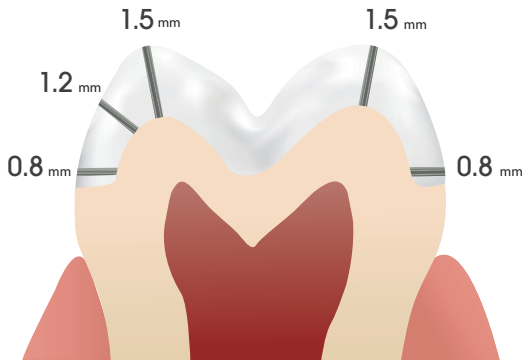

BUTT MARGIN


CEMENTATION

Conventionally cement following the manufacturer's instructions for use with zirconia restorations.

CONNECTOR CROSS SECTIONS

| | |
|-----------|---------------------|
| Posterior | ≥ 12mm ² |
| Anterior | ≥ 12mm ² |

BROWN DENTAL LAB

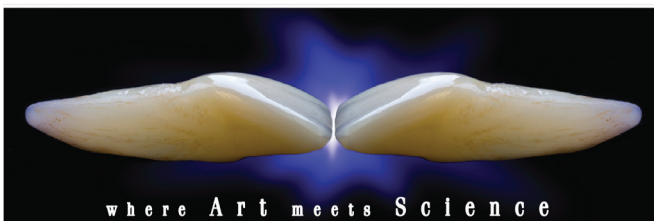

where Art meets Science

INDICATIONS FOR USE

| | | Imagine™ Zirconia |
|----------|------------------------|-------------------|
| Anterior | Crown | ✓ |
| | Bridge (up to 3 units) | ✓ |

| | | Imagine™ Zirconia |
|-----------|------------------------|-------------------|
| Posterior | Crown | ✓ |
| | Bridge (up to 3 units) | ✓ |
| | Inlay | ✓ |
| | Onlay | ✓ |

CHAIRSIDE ADJUSTMENTS

If chairside adjustments are needed:

- Fine diamond tools are recommended using a water-cooled, high speed hand piece.
- Use light pressure to avoid overheating and micro-cracking of the sintered body.
- Polish with diamond impregnated rubber polishers; maintain minimum wall thickness and connector cross sections.